Shower bench Grace

The shower bench Grace has two handgrips and a back plate (to mount it at a wall) and can be folded away and locked into an upright position. Height adjustable legs are included when extra support is needed.

The mounting plate has mounting holes to secure your shower bench to the wall. It should be fitted by a competent person and secured to a wall capable of supporting the vertical load and leverage affected upon it by the user. Vermeiren is not responsible for any damage or injuries caused by improper installation of this product.


! Maximum users weight: 130 kg

The shower bench Grace is designed for minimum maintenance and require the occasional wipe over with a soft cloth. Use a general neutral cleaning product with warm water. Do not use aggressive cleaning products that can cause damage or discoloration.

! Check the wall plate fittings on a regular basis.

To adjust the height, grab the horseshoe clip between forefinger and thumb and pull it out. Slide the leg out to the desired position and push the clip firmly back into the hole. NOTE: set the leg and rail height before securing the mounting plate to the wall. Ensure that both legs are of the same height.

! Be careful for your fingers when you fold away the shower bench.

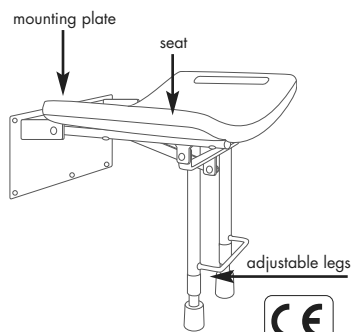Subject to technical alterations. © 2016

VERMEIREN
Opening up new horizons in mobility

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Vermeiren Nederland B.V.