13485:2016

Trive

English

Deutsch

Español

Français

Italiano

Dutch

Svenska

uplivin[®]

Uplivin UK Ltd.

Uplivin UK Ltd.

Suite 12/3D, Docklands Business Centre
10-16 Tiller Rd, London, E14 8PX, UK

info@uplivin.com
www.uplivin.com

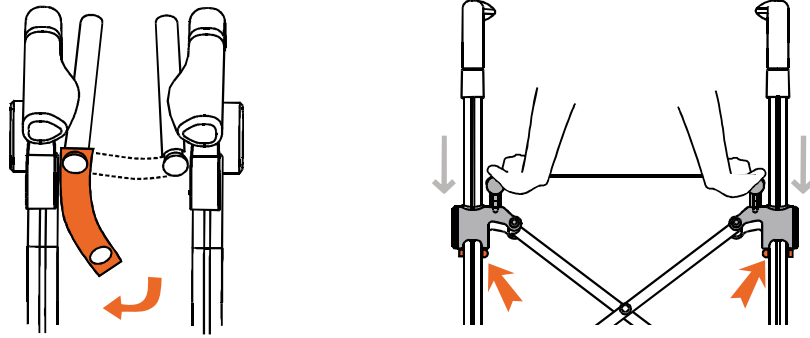
MedNet EC-Rep GmbH
Borkstrasse 10 · 48163 Muenster · Germany

Trive


English	User Manual.....	4
Deutsch	Bedienungsanleitung.....	8
Español	Manual de usuario.....	12
Français	Manuel d'utilisation.....	16
Italiano	Manuale d'uso.....	20
Dutch	Gebruiksaanwijzing.....	24
Svenska	Användarmanual.....	28


D


E







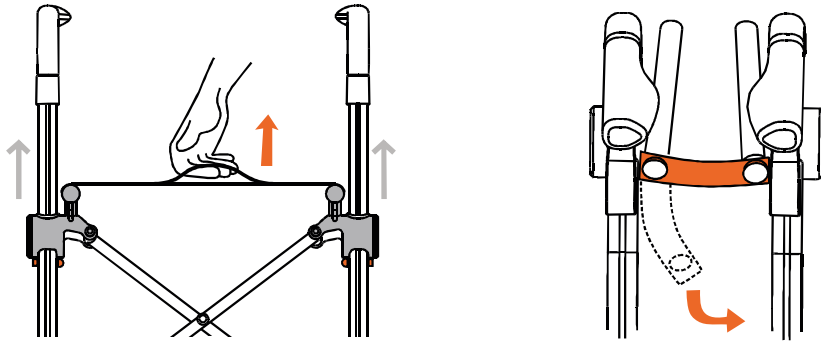
F

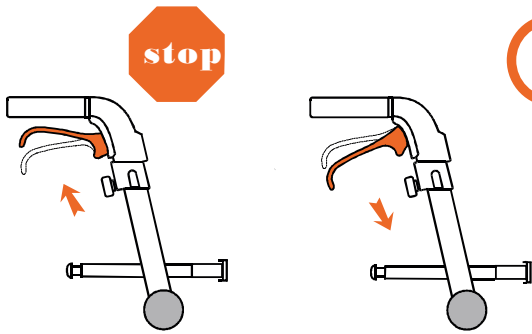H

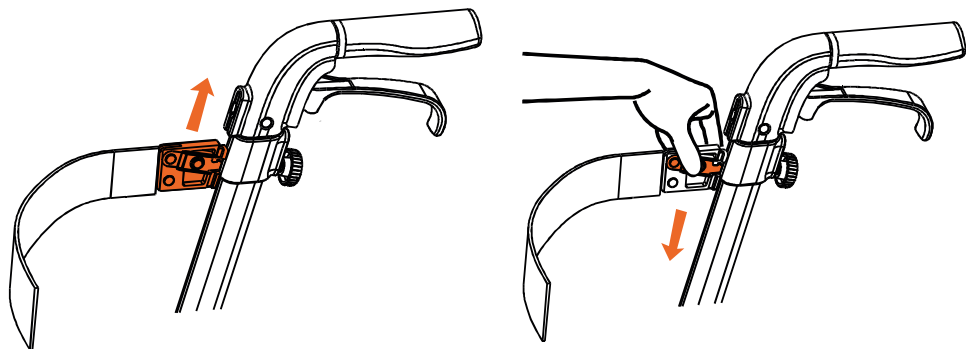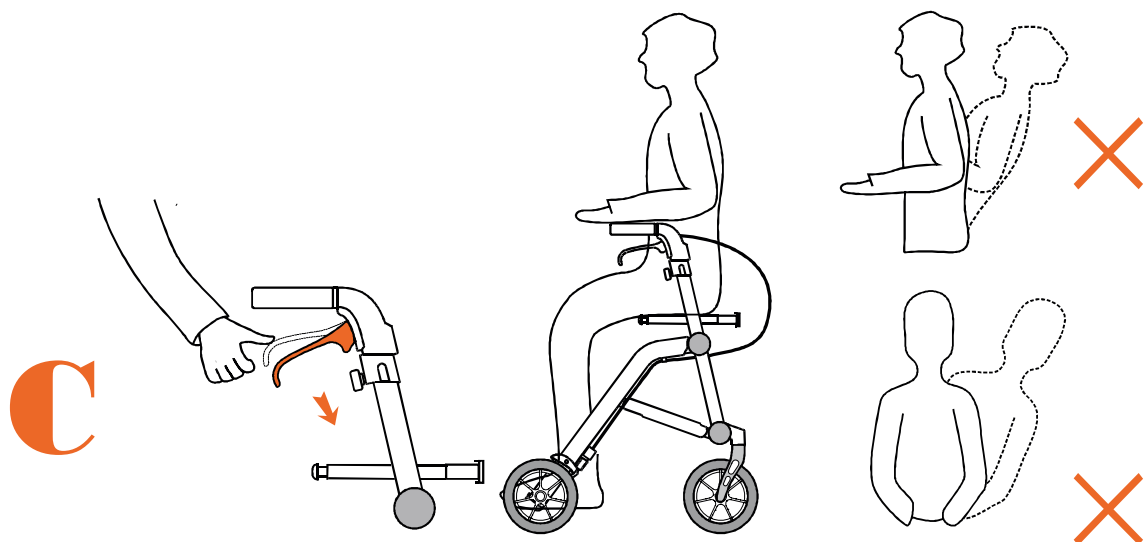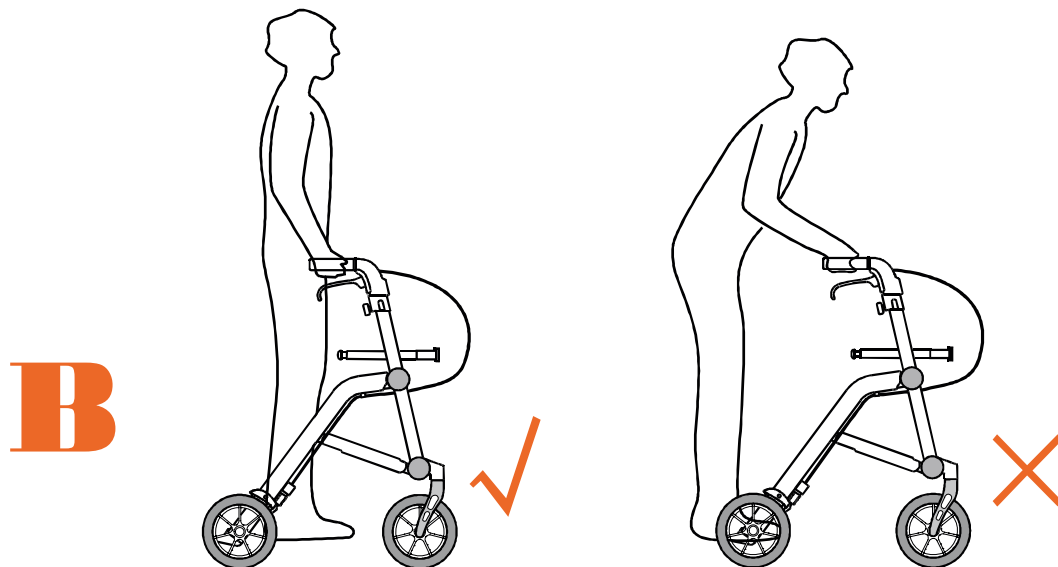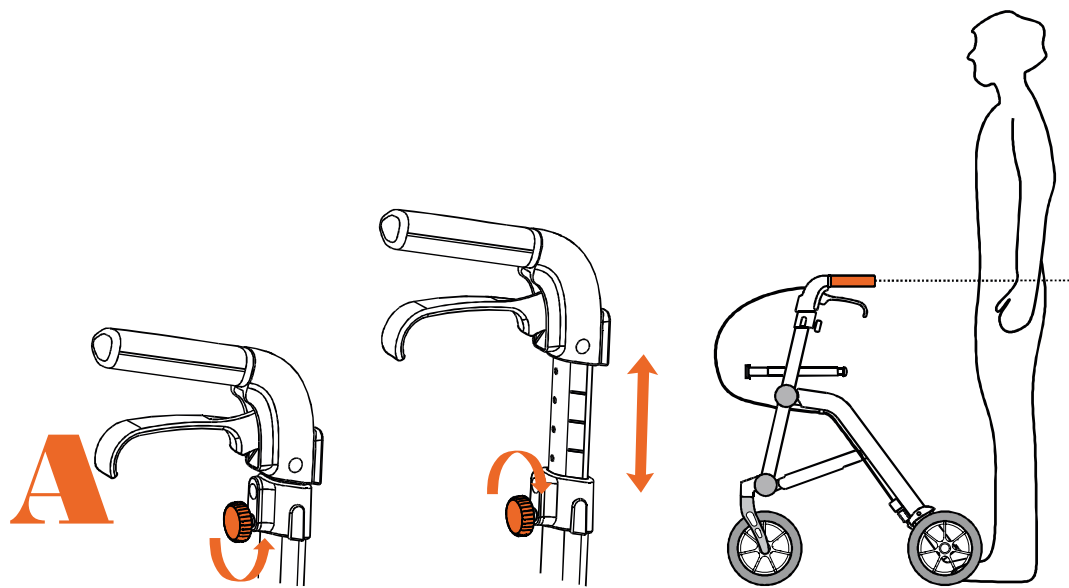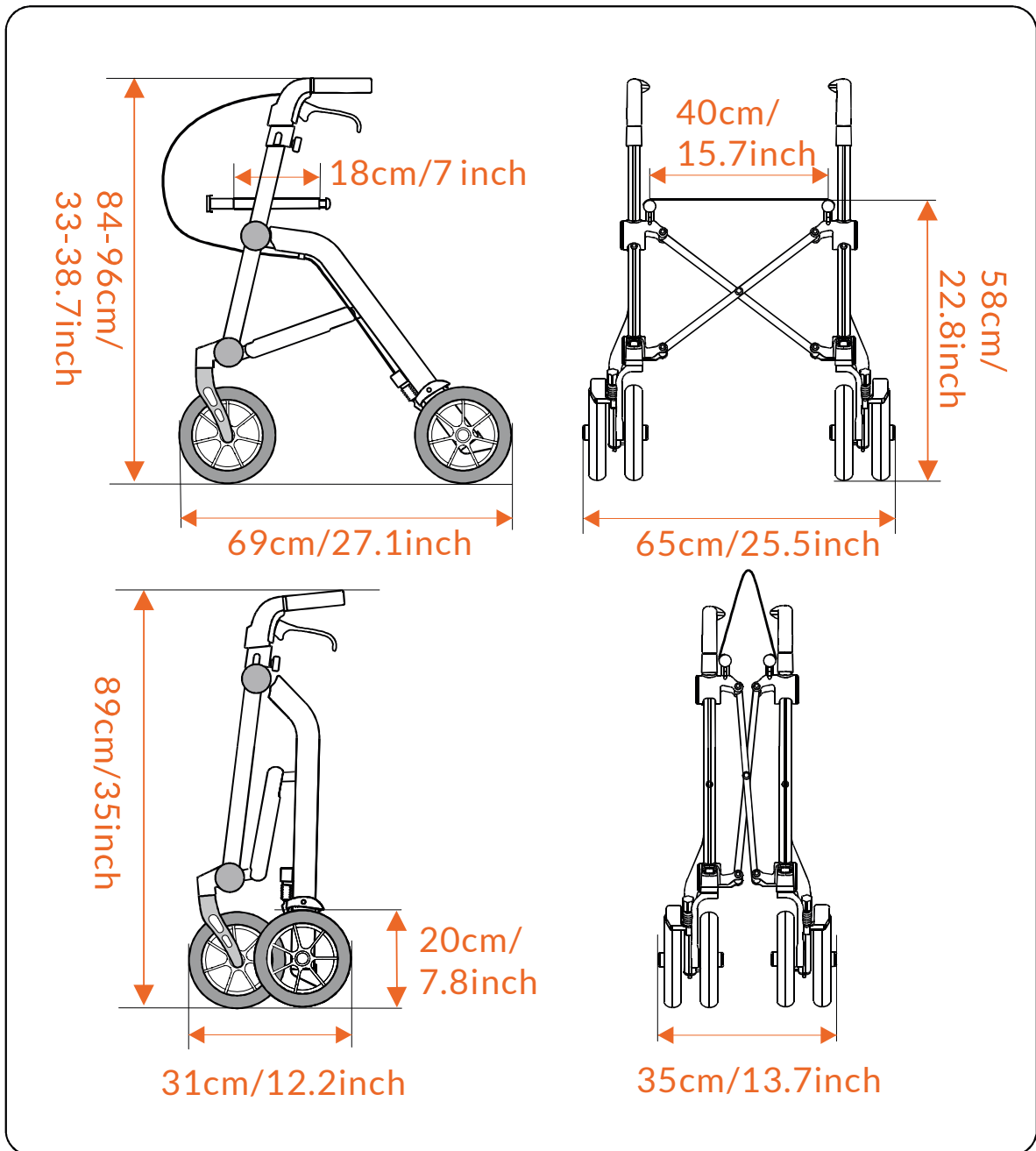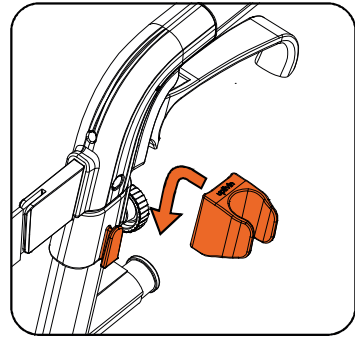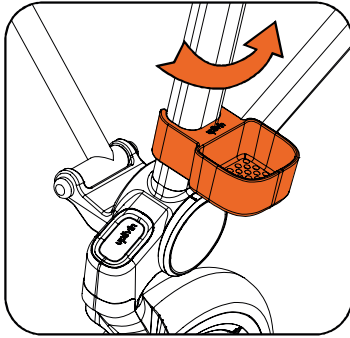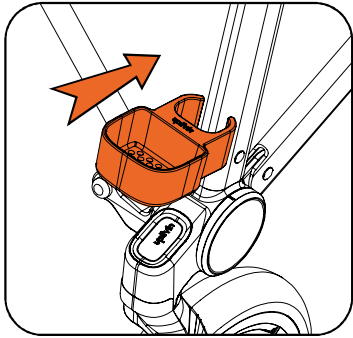Many congratulations for purchasing a Trive rollator which will improve your mobility making everyday life easier, safer and more enjoyable. Please note the following guidelines for the rollator's use.

The Trive rollator is designed for those with unsteady balance or poor walking ability to move around more confidently. It can be used both indoors and outdoors.

- When unfolding or folding the Trive rollator be careful that your fingers do not become trapped.
- Before using the Trive rollator, check that it is fully unfolded and fixed into position and that the brakes are functioning properly.
- Both brake handles must be applied at the same time whether using the driving brake or the parking brake.
- Before sitting on the Trive rollator's seat, the parking brake should be applied on both sides.
- The Trive rollator should not be pushed forward if either the parking brakes or driving brakes have been applied.
- The Trive rollator may only be used to sit on by persons under 125 kgs in weight. It should not be used to carry heavy goods or people.
- Children should be warned that the Trive rollator is not a toy.
- When the Trive rollator is exposed to extreme temperatures, care should be taken as certain parts may become very hot or very cold.

- Do not wear flowing or very long items of clothing when using the Trive rollator as material may become caught in the wheels resulting in injury to the user.

Maintenance: The Trive rollator should be cleaned using a general purpose detergent and tepid water. It should not be stored in damp or very hot conditions.

Warranty: Our two-year warranty does not cover wear and tear of certain parts such as the rollator's wheels and cables.

A

Handle height adjustment

For health and safety purposes, it is important that the handle height of the Trive rollator is adjusted to the correct height for the user. To achieve this, the user should stand between the handles of the Trive rollator with their arms hanging loosely by their sides, elbows bent naturally and with the palms of the hands turned inwards. The position of the wrists marks the correct height of the handle for the user.

To make any adjustment to the handle height, untighten the two adjustment screws, change the height to the required position and tighten the screws again.

Please note that in order to achieve the correct posture and evenly distributed body weight, both handles must be adjusted to the same height.

B

Walk

While walking with the Trive rollator, the user should remain upright, holding the handles close to the body.

C

Seat

The parking brakes must be applied before sitting on the Trive rollator to prevent it from moving while seated. When in the seated position, the user's back should always be against the rollator's shopping bag.

Warning: In the interests of safety, when sitting on the Trive rollator the user should not lean over its sides or lean backwards.

D

Unfolding the Trive rollator

Release and push down the lock clamp at the side of the seating bar.

E

Folding the Trive rollator

Lift the seat in the middle, push the handles together and secure with the lock clamp.

F

Driving and parking brakes

- 1.To apply the driving brakes when the Trive rollator is in use, simply pull the handle levers up.
- 2.To apply the parking brakes, push the handles down fully. To release, pull the handles back up.

G

Adjustments to the brakes

The brakes may be adjusted using the adjustment screw to tighten or loosen as required.

H

Backrest

1. To install, slide one end of the backrest up the handle until a light click sound is heard. Repeat with the other end of the backrest on the opposite handle.
2. To uninstall, press and hold the spring levers while lowering both sides of the backrest to the bottom of the handles.

Specifications

Handle height adjustable	84 to 96cm
Unfolded Width*Length	65*69cm
Folded Width*Length	35*31cm
Seat Height	58cm
Weight	6.5kg
Wheel diameter	20cm
Max user weight	125kg
Max shopping bag load	5kg

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