es

ca

en

FR

DE

IT

DU

# CARLETT

care

## Lett900

# CE

español

# Instrucciones

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english

# Instructions

Thank you for purchasing this Carlett product. Remember to keep the receipt, because it is your only proof of guarantee if the product is defective.

Any defect arising from misuse of the product by the user and the possible normal deterioration of certain components (wheels and textile components) due to wear and tear of the product are excluded from the guarantee.

Before using the trolley, please read the instructions for its proper use carefully.

We hope that you enjoy this product, which has been designed to help improve its users' quality of life.

Thank you!


# The Lett900 - for shopping and sitting comfortably

en

The Lett Model **900** allows the user to go shopping and walk with the peace of mind of being able to sit down and rest whenever necessary.


It has a safety or parking brake, which is essential for using the trolley as a seat. To activate the safety brake, pull the 2 yellow handlebar buttons. To take the brake off and start moving again, press the 2 handlebar levers. (see image no. 1).

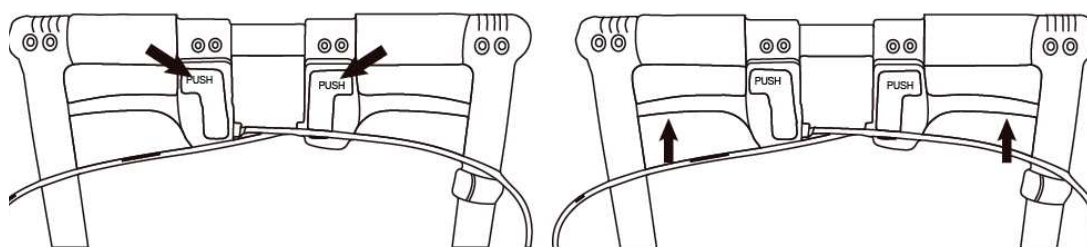*Image 1. Safety brake*

It incorporates an instant braking system which is activated by pulling the 2 handlebar levers until the trolley stops. To disable it and/or continue walking, simply stop pulling the handlebar levers(see image no. 2).


The product also has a **system for adjusting**

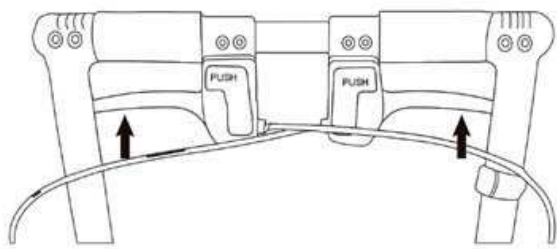*Image 2. Instant brake*

rotation or fully rotating (360°). To change the degree of rotation, press the adjustment mechanism located in the front wheel and adjust it as required (see image no. 3).

**the degree of rotation** of the two front swivel wheels according to your needs. The wheels can be completely fixed, with a 90°

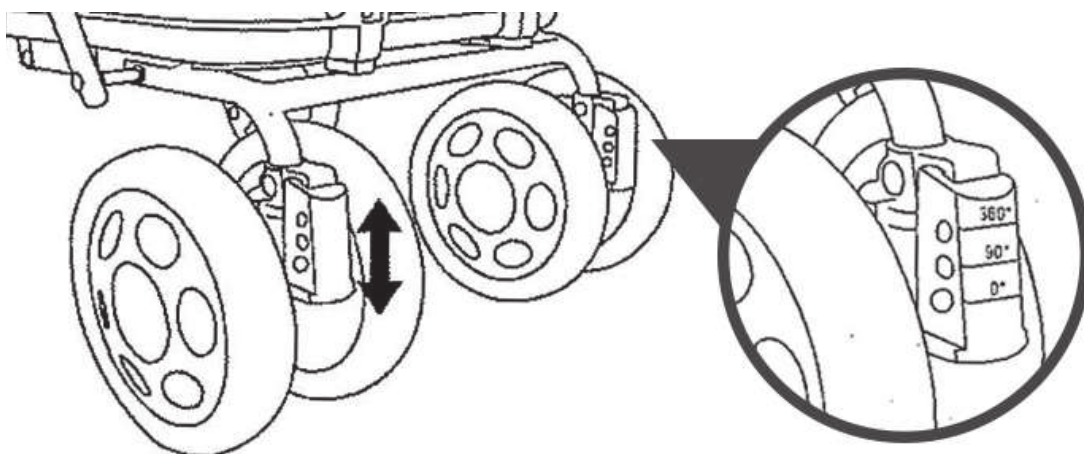*Image 3. Rotating wheels*

It has a **height adjustment mechanism for the handlebar** with three positions. To change the height, remove the blockers from the holes located on the left and right of the handlebar, move it to the desired height and secure it again by introducing the blockers into the correct holes on both sides (see image no. 4).

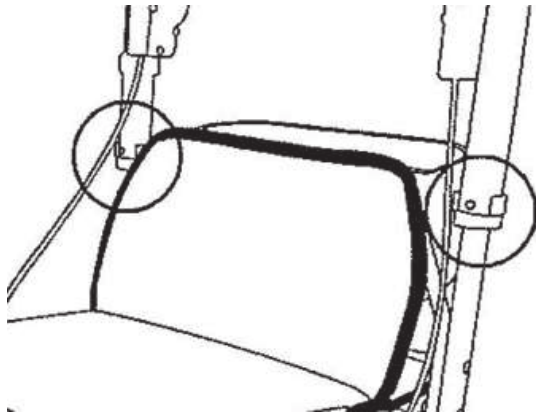
Image 4.  
Adjustable handlebar

Image 5. Umbrella support

It contains a **rack for umbrellas**, which can be fastened using the strap (see image no. 5).

Image 6. Folding system

To fold up and store the product, you only need to pull the seat cushion upwards (see image no. 6).

It has a removable inner bag with handles which is easy to clean.

## Material

- Chassis: aluminium
- Textile components: Polyester

- Wheels: EVA material (this material means that the wheels are silent, even on rough surfaces).
- Tyres: polypropylene

## Specifications

- Unfolded product: W52 x L58 x H 84-91 cm
- Folded product: W52 x L38 x H 84-83 cm
- Seat size: W 35 x L 28 cm
- Maximum seat load: 100 kg
- Maximum load in the removable inner bag: 15 kg
- Net weight of the chassis: 6.2 kg

## Maintenance

**Use a dry cloth for everyday maintenance.** If you need to remove any stains, we recommend using a neutral detergent and wiping off the stain manually.

In order to lengthen the life of the product, in the event of rain or contact with water, you should pat the chassis dry with a dry cloth.

Do not modify or dismantle the product because this can lead to malfunctioning, and may even make it no longer safe to use.

en

## Recommendations

This rollator is deemed to meet the Essential requirements and relevant provisions of EC Directive: Medical Devices Directive 93/42/EEC. The company cannot be held responsible for any other use made of the product.

Do not move the trolley when a person is sitting on the seat.

**The maximum weight that the bag holds is 15 kilograms.** Do not overload the bag even when the maximum weight has not been reached. Always put the heavier items at the bottom of the bag.

Take special care when using the product on a ramp or a street with a slope, especially if the trolley is loaded. Remember to use the handbrake to control the trolley.

Take special care when using the trolley on wet or slippery ground.

Check that the safety or parking brake is on before using it as a seat.

This product is not a toy. Do not allow children to handle and/or climb on it.

Do not use the trolley to transport pets.

Do not keep the trolley in places that are damp, very hot or exposed to direct sunlight, as this may shorten the life of the product.

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