## PEDALIER A AFFICHAGE LCD

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## PEDAL EXERCISER WITH LCD DISPLAY

*Instructions manual*

REF 534012

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Pile : 1 pile LR44 (fournie)

This pedal exerciser can be folded and easily stocked. It is recommended to help blood circulation and improve muscular development. It is a fun and entertaining way to do cardiovascular exercises. Placed on a table or even the ground, you can use it with your arms or legs.

### ASSEMBLING

Carefully remove the pedal exerciser from its packaging. Unfold firmly each leg of the exerciser until you hear a click for the locking.

The pedal exerciser is ready to use.

To fold the pedal exerciser, pull on the pin rings and fold down each leg of the pedal exerciser.

### UTILISATION

Put yourself in a comfortable position and make sure there are no obstacles for arm or leg movements.

Put your feet in the pedal straps and start to exercise.

To increase or reduce the pedaling resistance, adjust the tightness of the knob.

The pedal exerciser is resistant to a maximum weight of 70kg.

#### CAUTION

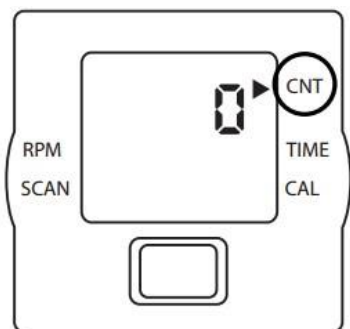
Ensure that the device is put on an even surface. Always start exercising with a low resistance and gradually increase it.

The LCD display turns itself on as soon as the pedalling starts.


The display shows various features.

### DISPLAYED FEATURES


To switch on the display, press the red button once or start pedaling. The display switches itself off after 4 minutes of non-utilisation. All features are activated when the pedaling starts and will pause when the pedaling is stopped. To switch from one feature to another, press successively on the red button.

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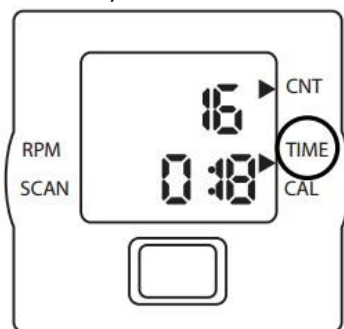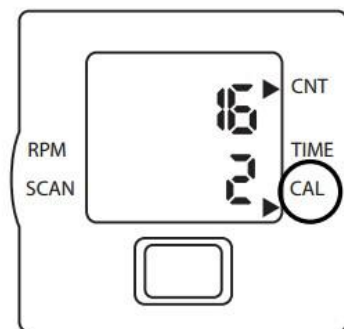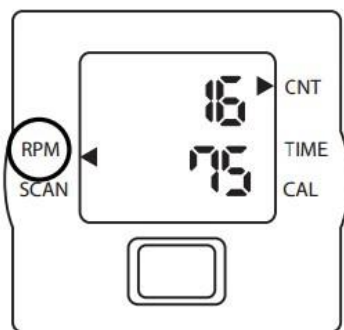
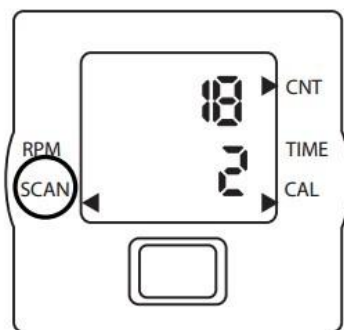
SCAN:  
Changement (TIME>CNT>RPM>CAL)

CNT	Number of pedal turns realised
TIME	Exercise time
CAL	Estimated calories burnt
RPM	Rotations per minute
SCAN	Changing (TIME>CNT>RPM>CAL

## SPECIFICATIONS

Time from 00:00 to 99:59 min

Count from 0 to 9999 turns

Turns/min from 0 to 999 rpm Calories from 0 to 9999Kcal

## FEATURES

**TIME** – Press the red button several times to display the TIME. The countdown will start at the beginning of the pedaling and will stop when the pedaling stops.

**TURNS PER MINUTES** – Press the red button several times to display the RPM. The rotation speed per minute will display at the beginning of the pedaling and will stop when the pedaling stops.

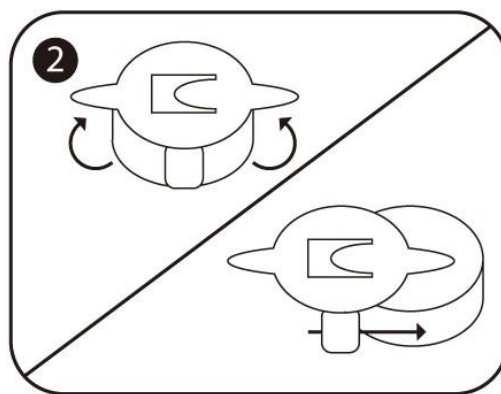
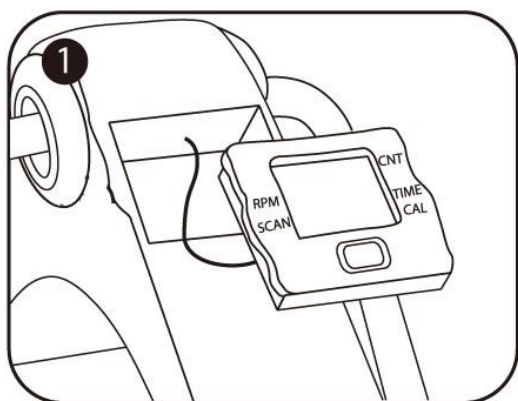
**CALORIES** – Press the red button several times to display the CAL. The estimated count of burnt calories will display from the beginning of the pedaling and will stop when the pedaling stops.

**TURN COUNT DOWN** – The cursor is always placed on CNT, whatever the feature selected. The number of pedalling turns realised is always displayed.


**SCAN** – Press the red button several times to display the SCAN. The simultaneous countdown of all features starts from the beginning of the pedalling (TIME, CALORIES, NUMBER OF TURNS, ROTATIONS PER MINUTE)

To reset all features, press 4 to 5 seconds on the red button. All data will be erased.


## CHANGE OF BATTERIES - LR44 (provided)

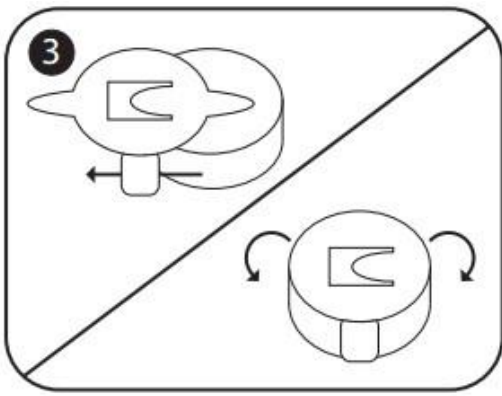- 1) Use a nail or a small tool to
- 2) At the back of the display, lift to separate the display from its lodging, and slide the battery, then remove carefully without pulling on the sides and cables.

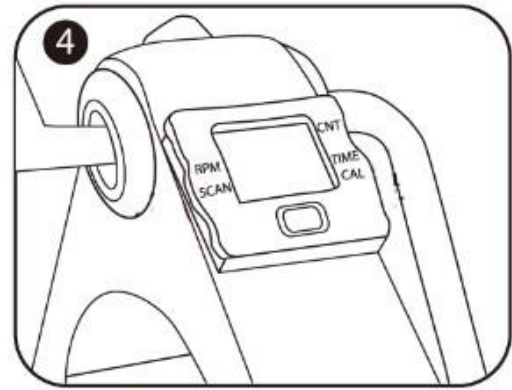3) Insert a new battery 1.5V AG13

4) Replace the display box carefully placing the + facing down, then close.

## DISPOSAL, RECYCLING

- Please follow instructions from local environmental authorities.
- Dispose of the packaging and pedal exerciser separately following the local environmental requirements.

### CAUTION

This product is designed for private use only.

## TECHNICAL FEATURES

Maximum load : 70kg

Dimensions: H23.5 x Depth 49.5 x Width 37cm (including pedals)

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